10*15cm

Tolaccea

3D

SHOULDER ICE PACK COOLING GEL BRACE

PRODUCT MANUAL

HOT PRODUCTS

Elbow & Knee Compression Sleeve Ice Pack
Colour: Black, Blue
Size: S, M, L, XL; Unisex

Ankle Foot Ice Pack Wrap
Colour: Black, Blue
Size: L, XL; Unisex

Stress Relief Balls
Color: Black, Blue, Light Blue
Size: 2.4in

Ankle Foot Ice Pack Wrap
Colour: Black, Blue
Size: L, XL; Unisex

Half Finger Arthritis Compression Ice Glove
Colour: Black
Size: S, M; Unisex

Reusable Gel Cold Therapy Sock
Colour: Black
Size: M, L; Unisex

Thumb Wrist Ice Pack with Adjustable Wrist Strap
Cour: Black
Size: One size fits all; Unisex

Foot Ice Packs Big Toe for Pain Relief
Colour: Black
Size: One size fits all; Unisex

Wrist Ice Pack Wrap
Colour: Black
Size: S, M; Unisex

Foot Ice Pack for Toes
Colour: Black
Size: One size fits all; Unisex

3D SHOULDER ICE PACK COOLING GEL BRACE
Colour: Black
Size: Universal, L Unisex

- EN -

DESCRIPTION

Experience effective pain relief with advanced 3D thickened gel technology, providing long-lasting cold therapy. For extended cooling, add the included frozen gel packs for enhanced compression and double the cooling effect. Perfect for reducing swelling, muscle pain, injuries, and aiding post-surgery recovery.

HOT THERAPY INSTRUCTIONS

- Microwave the product for 50 seconds to 1 minute at 700 watts. Test for comfort.
- If additional heat is needed, flip the product and heat in 10-second intervals, up to 1 minute per side.

COLD THERAPY INSTRUCTIONS

- Freeze the shoulder gel brace at -20°C for 1-2 hours.
- For enhanced cooling, insert the included liquid gel packs.
- Wipe off any moisture before applying for 10 minutes or as needed.

CARE INSTRUCTION:

Use damp cloth to wipe down. Lay flat and air dry. Do not wash, bleach, dry clean or iron.

WHAT IN BOX:

Shoulder Gel Brace × 1 / Extra Frozen Gel × 1 / Velcro Straps × 1

WARNING!

1. Limit cold/hot therapy to 30 minutes to prevent frostbite or burns. If not cold enough, refreeze at -20°C.
2. Do not microwave over 45 seconds. Check temperature before use; cool if too hot to avoid burns.
3. Repeated heating may dry out the gel, reducing lifespan. Monitor while microwaving to avoid overheating. Air dry if damp—never use a dryer.
4. For external use only. Avoid areas with poor circulation, open wounds, or infections unless advised by a professional.
5. If gel contacts skin, wash thoroughly. Seek medical advice for injuries, and discontinue if discomfort persists.
6. Store in therapy wrap when not in use to prevent condensation. Ensure dry before storage.
7. Keep away from children and pets.

AFTER SALE

Tolaccea guarantees 1 year warranty for the quality problem. Any questions please freely email us.service@tolaccea.com.

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Company Name: REPLAB CONSULTING LTD