10*21cm

Tolaccea

OPERATION INSTRUCTION

Hot & Cold Therapy Wrist Compression Sleeve

Tolaccea Hot Selling Gel Ice Packs

Tolaccea Ankle Ice Pack Wrap PRO

Colour: Black
Size: L, XL; Unisex

Elbow & Knee Compression Sleeve Ice Pack

Colour: Black - Blue
Size: S - M - L - XL; Unisex

Shoulder Ice Pack Rotator Cuff Cold Therapy

Colour: Black
Size: One size fits all; Unisex

Half Finger Arthritis Compression Ice Glove

Colour: Black
Size: S - M - L; Unisex

Thumb Wrist Ice Pack with Adjustable Wrist Strap

Colour: Black
Size: One size fits all; Unisex

Wrist Ice Pack Wrap

Colour: Black - Blue
Size: S - M - L; Unisex

Ankle Foot Ice Pack Wrap

Colour: Black - Blue
Size: L - XL; Unisex

Ankle Foot Ice Pack Wrap

Colour: Black - Blue
Size: L - XL; Unisex

Reusable Gel Cold Therapy Sock

Colour: Black
Size: M - L; Unisex

Foot Ice Packs Big Toe for Pain Relief

Colour: Black
Size: One size fits all; Unisex

Foot Ice Pack for Toes

Colour: Black
Size: One size fits all; Unisex

English

Functions:

1. Tolaccea hot & Cold therapy wrist compression sleeve provides 360° coverage.
2. The flexible ice pack is in ergonomic designs perfect for wrist.
3. This ice pack is reusable, aslo microwave and freezer safe, allowing you to use hot or cold therapy depending on your needs.
4. Wrist compression sleeve is made of non-toxic material, latex-free, and leak proof, with no smells after use.

Operation Instruction:

- Cold packs**
- Place the sleeve in a refrigerator at -20° for 1-2 hours for immediate use. If colder temperature needed, frozen it longer with lower temperature.
 - Take out the sleeve from freezer, wipe off the moisture and pull it over the required area.
 - Wear sleeve onto the treatment area of the body and keep in place for approximately 15-20 minutes.

Hot therapy

- Start with room temperature pack, place the sleeve in microwave with a clean plate.
- Microwave the sleeve for 10 seconds, if more heat is needed microwave in 5 second increments.
- Wear sleeve onto the treatment area of the body and keep in place for approximately 15-20 minutes.

How to clean it

Hand wash with damp towel. Do not submerge sleeve in water. Lay flat and air dry, do not machine wash.

WARNING!

1. Limit cold/hot therapy to 30 minutes to prevent frostbite or burns. If not cold enough, refreeze at -20°C.
2. Do not microwave over 45 seconds. Check temperature before use; cool if too hot to avoid burns.
3. Repeated heating may dry out the gel, reducing lifespan. Monitor while microwaving to avoid overheating. Air dry if damp—never use a dryer.
4. For external use only. Avoid areas with poor circulation, open wounds, or infections unless advised by a professional.
5. If gel contacts skin, wash thoroughly. Seek medical advice for injuries, and discontinue if discomfort persists.
6. Store in therapy wrap when not in use to prevent condensation. Ensure dry before storage.
7. Keep away from children and pets.

After Sale

Tolaccea guarantees 1 year warranty for the quality problem. Any questions please freely email us: service@tolaccea.com

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UK REP Company Name: REPLAB CONSULTING LTD